

Account: \_\_\_\_\_ Date: \_\_\_\_\_

Farm Name: \_\_\_\_\_ Contact Person: \_\_\_\_\_

Street: \_\_\_\_\_

City: \_\_\_\_\_ State: \_\_\_\_\_ Zip: \_\_\_\_\_

Phone: \_\_\_\_\_ E-mail: \_\_\_\_\_

**Analysis:**  Pregnancy  Johne's    **Species:**  Cattle  Goat  Sheep    **Reorder:**  10  50

**Direction for using the 50 Vial Test Kit**

1. Order kits by e-mailing supply@dairyone.com or check box for re-order. Kits can also be ordered through the online store at www.dairyone.com
2. Obtain a milk sample for each cow to be tested in one of the pre-preserved sample vials. To ensure a usable sample each vial should be at least 1/3 full.
3. Close the lid tightly and mark the sample vial with the vial number on top with a permanent marker. Record the ID of the animal with the vial number on the submission form. Record DPB + DIM information.
4. Place vials in the provided whirlpak bag. Place whirlpak bag in the bubble envelope and refrigerate samples until ready to ship.
5. Place submission for in the bubble envelope (not in the whirlpak bag) and place label on the envelope.
6. Free shipping for 10 or more samples.

**Send to: Dairy One Animal Health Diagnostic Laboratory  
 730 Warren Road,  
 Ithaca, New York 14850**

**Disclaimer** - By using this form, the customer confirms that they understand the following:

- The laboratory cannot test samples that are separated or spoiled; the laboratory may not notify customer of untestable samples prior to testing other samples in their submission.
- Days Post Breeding and Days In Milk information is necessary to determine the suitability of samples for milk pregnancy testing and to provide proper interpretation of results. By not providing DPB and/or DIM information that conforms to validity criteria, the validity of the test may be affected. Cows and goats must be >28 DPB & >60 DIM. Sheep must be >60 DPB & >80 DIM.
- If a Sampled Date is not provided, the date of the test will be used.
- Simplified test results in the form of a .csv file may be sent to an unsecure digital area.
- The AHDL uses the IDEXX Alertys Milk Pregnancy Test & its Optional Interpretation for milk pregnancy testing. The AHDL uses IDEXX MAP (Mycobacterium Paratuberculosis Antibody) Test for Johne's Disease testing.
- For questions about analysis, call 1-800-344-2697 ext 7615

Vial #	Animal ID Number	Days Post Breeding	Days in Milk
1			
2			
3			
4			
5			
6			
7			
8			
9			
10			
11			
12			
13			
14			
15			

Vial #	Animal ID Number	Days Post Breeding	Days in Milk
16			
17			
18			
19			
20			
21			
22			
23			
24			
25			
26			
27			
28			
29			
30			

Vial #	Animal ID Number	Days Post Breeding	Days in Milk
31			
32			
33			
34			
35			
36			
37			
38			
39			
40			
41			
42			
43			
44			
45			
46			
47			
48			
49			
50			
51			
52			
53			
54			
55			
56			
57			
58			
59			
60			
61			
62			
63			
64			
65			

Vial #	Animal ID Number	Days Post Breeding	Days in Milk
66			
67			
68			
69			
70			
71			
72			
73			
74			
75			
76			
77			
78			
79			
80			
81			
82			
83			
84			
85			
86			
87			
88			
89			
90			
91			
92			
93			
94			
95			
96			
97			
98			
99			
100			