



Dairy One



Milk Pregnancy and Johne's Sample

Submission Form

Order kits by e-mailing supply@dairyone.com or check box for re-order. Kits can also be ordered through the online store at www.dairyone.com. For questions about analysis, call 800-344-2697, ext 7615.

Contact Person: _____ Date: _____

Farm Name: _____ Account No.: _____

Street: _____

City: _____ State: _____ Zip: _____

Phone (area code): _____ email: _____

Analysis: Pregnancy Johne's Species: Cattle Goat/sheep Reorder: 10 50

Directions for the 50 Vial Test Kit

- Cows can be tested 28 days post breeding and 60 days post calving.
- Obtain a milk sample from each cow to be tested in one of the pre-preserved sample vials. To ensure a usable sample each vial should be at least 1/3 full.
- Close the lid tightly and mark the sample vial with the vial number on top with a permanent marker. Record the ID number of the cow with vial number on the submission form.
- Place vials in the provided Ziploc bag. Place Ziploc bag in the bubble envelope and refrigerate samples until ready to ship.
- Place submittal form in the bubbled envelope (not in the Ziploc bag) and place label on the envelope.

(Free shipping for 10 or more samples):

Dairy One Animal Health Diagnostic Lab
730 Warren Road
Ithaca, New York 14850

Vial #	Animal ID Number	Days Post Breeding	Days Post Calving
1			
2			
3			
4			
5			
6			
7			
8			
9			
10			
11			
12			
13			
14			
15			

Vial #	Animal ID Number	Days Post Breeding	Days Post Calving
16			
17			
18			
19			
20			
21			
22			
23			
24			
25			
26			
27			
28			
29			
30			

Vial #	Animal ID Number	Days Post Breeding	Days Post Calving
31			
32			
33			
34			
35			
36			
37			
38			
39			
40			
41			
42			
43			
44			
45			
46			
47			
48			
49			
50			
51			
52			
53			
54			
55			
56			
57			
58			
59			
60			
61			
62			
63			
64			
65			

Vial #	Animal ID Number	Days Post Breeding	Days Post Calving
66			
67			
68			
69			
70			
71			
72			
73			
74			
75			
76			
77			
78			
79			
80			
81			
82			
83			
84			
85			
86			
87			
88			
89			
90			
91			
92			
93			
94			
95			
96			
97			
98			
99			
100			